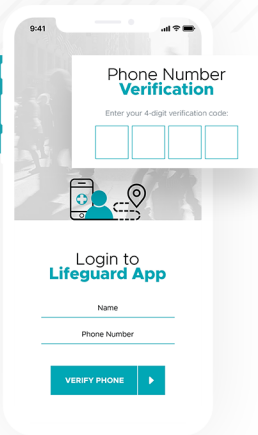
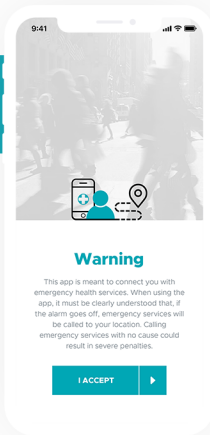
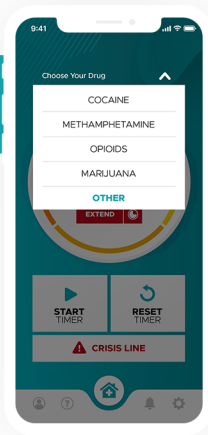


**Step 1**


Enter your name, contact number, and the **4-Digit Verification Code** that is sent to you via text message.

**Step 2**


Read through the warning message, and tap the **I Accept** button.

**Step 3**


**Select** the drugs you will use and tap the **Start** button.

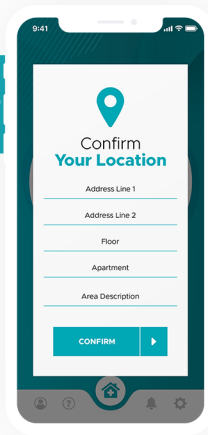**Step 4**


**Confirm your address**, and provide additional location details if necessary.

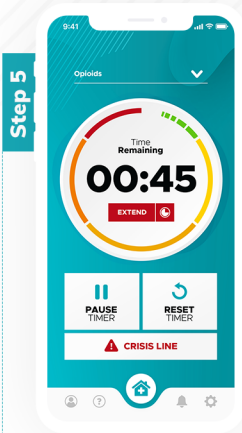

▶ The timer starts running. If you need to stop it for any reason, tap the **Pause** button.

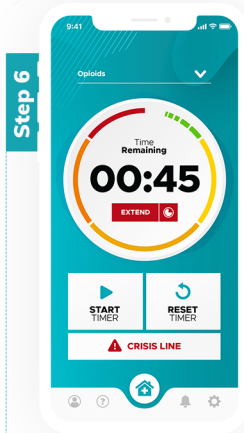

▶ When you are ready to resume the timer, tap the **Start** button.

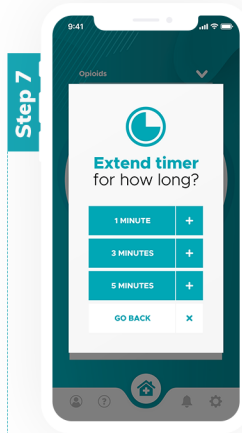

▶ To increase the length of the timer, tap the **Extend** button and select a time from the menu.



▶ At 10 seconds remaining, an alarm will ring. When the timer ends, tap the **Stop** button.